  
**Topshelf**  
*Manufacturing, LLC*  
1851 East Paradise Road, Suite A  
Tracy, CA 95304  
Phone: 866.592.0488 • Fax: 209.834.8832  
topshefforthopedics.com

# Tri-Panel Knee Immobilizer

## Instructions For Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.


# Tri-Panel Knee Immobilizer

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is critical to the proper function of the brace.

**Indications:** Ideal for post-operative and post-injury immobilization of the knee.

### Application Instructions:

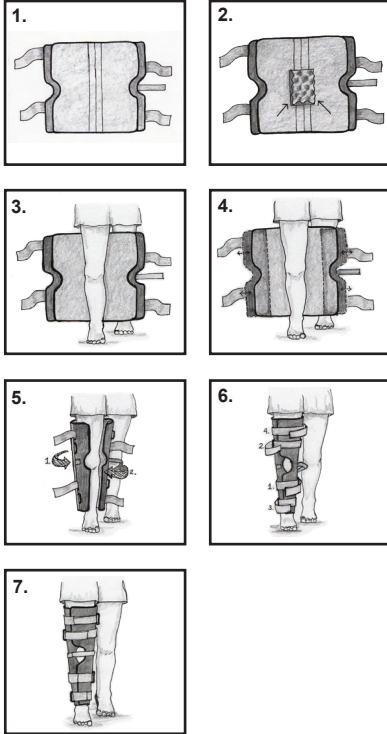

1. Unfasten all straps and open the immobilizer completely.
2. Affix the foam to the inside of the immobilizer at the center of the brace.
3. Position the leg over the affixed foam where the knee bends.
4. Adjust the outer detachable panels by removing them from the immobilizer and re-positioning them to obtain a more accurate fit to the leg anatomy. Affix them back to the immobilizer body.
5. With the kneecap centered secure the first Velcro strap just below the knee.
6. Secure the remainder of the straps in an alternating fashion until they are all completed.
7. A properly applied Tri-Panel Knee Immobilizer.

#### Use and Care:

Hand wash the immobilizer with mild soap and water and lay out to dry. Do NOT machine dry.

#### Notes:

For single patient use only.

#### Warning:

This product is to be used under the supervision of a medical professional. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

#### Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of three months from the date of purchase.

#### Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.