

Knee Sleeve

Instructions For Use

Indication:
For General
Support & Compression

Please read these instructions completely and carefully before applying the sleeve. Correct application of the sleeve is important for proper function.


Manufactured by:




Tracy, CA 95304
Phone: 866.592.0488 • Fax: 209.834.8832

APPLICATION AND FITTING INSTRUCTIONS

Grasp edges of the sleeve at the larger opening. Bring the sleeve onto the knee so that the patella (knee cap) is centered within the sleeve.

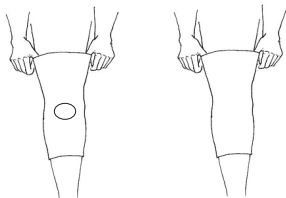

If buttress is included:

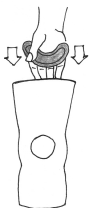

1. Remove the buttress by reaching inside the brace and gently pull away the buttress from the inside of the brace.

2. If the buttress is required it may be positioned based on the patient's needs. Slide the buttress with hook forward to allow it to engage on the loop, which is on the inside of the brace around the patella (knee cap) opening.



CARE INSTRUCTIONS

Hand wash in cool water utilizing mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.