



# Knee Immobilizer, Universal

## Instructions For Use

Please read these instructions completely and carefully before applying this brace.


Correct application of the brace is important for proper function.



## KNEE IMMOBILIZER, UNIVERSAL APPLICATION INSTRUCTIONS


Before applying the brace, please read these instructions completely and carefully.  
Correct application is important to the proper functioning of the brace.

(loosen all straps prior to application)

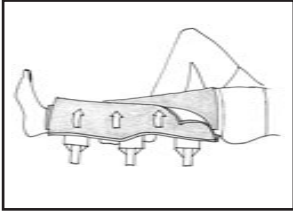


**A.** Start by removing the medial & lateral side bars, and undo all straps. Then, with patient supine, strap foam padding around involved leg, centering posterior stay in the popliteal area.


Excess foam padding can be trimmed away if needed.

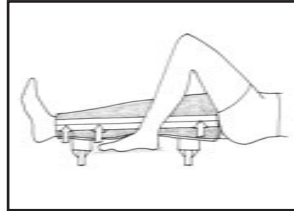

**B.** With the leg in full extension, reattach and reposition medial side bar into place by pressing it back onto foam padding. Center medial side bar anteriorly and posteriorly. Repeat for lateral side bar.

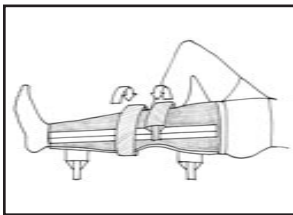

**C.** Secure the strap(s) closest to the knee. The strap(s) will circle the leg and attach to any part of the foam padding.

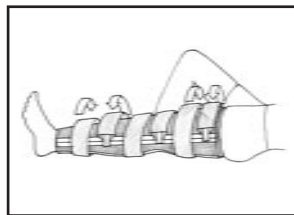

**D.** Wrap and attach the bottom strap(s) at the calf, then the top strap(s) at the thigh.

**Note:** For patients with an unusually large degree of varus or valgus, the medial and lateral side bars can be bent to accommodate the position. Bend each set of side bars an equal amount above and below the midpoint toward the varus or valgus bend in the leg. Side bars should be bent in small increments until the proper adjustment is achieved.

**WARNING: IF YOU EXPERIENCE ANY PAIN OR SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.**

### WARRANTY

Top Shelf guarantees this product to be free of defects in material and construction for a period of six months from date of purchase.