

APPLICATION AND FITTING INSTRUCTIONS

- 1.** Center the collar under the chin.
- 2.** Wrap both ends of the collar around the back of the neck and secure the hook and loop.
- 3.** Adjust for comfort.

CARE INSTRUCTIONS

Hand wash in cold water using mild soap. Rinse thoroughly and air dry. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

WARNING

If you experience pain, swelling, or sensation changes as a result of wearing this brace, consult your healthcare provider immediately.

Neoprene braces and supports should not be worn by individuals with known allergies to neoprene products or susceptibility to dermatitis.

Federal Law (USA) restricts this device to sale by or on the order of a licensed healthcare professional.

Cervical Collar

Instructions For Use

Indication:
Immobilization of the neck

Please read these instructions completely and carefully before applying this brace. Correct application of the brace is important for proper function



1851 East Paradise Road, Suite A
Tracy, CA 95304

Phone: 866.592.0488 • Fax: 209.834.8832
topshelforthopedics.com

P/N: 025091 Rev: A