

topshelforthopedics.com
Phone: 866.592.0488 • Fax: 209.834.8832
Tracy, CA 95304
1851 East Paradise Road, Suite A
Topshelf
Manufacturing, LLC

BOA with Slim Panel XTR

Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this product.



BOA with Slim Panel XTR Instructions for Use

Fitting, Use, and Care Instructions

Please read these instructions completely and carefully before applying this product.
Correct application is important for proper function of this product.

Indications: Acute and chronic low back pain, Post-operative support, Disc herniation.

Contraindications: Pregnancy and circulation, Pulmonary, Cardiovascular or Skeletal conditions which have risk to be made worse as a result of compression and/or pressures.

- A.** Position and fasten the pull-handle as close as possible to the compression panels, then fully extend the cording between the 2 halves of the brace.
- B.** Place the slim panel chairback on a flat surface with the mesh interior facing down. Place the plastic inserts into the elastic pockets and secure the compression panels over the chairback slim panel. Ensure the compression panels are centered over the panel for maximum comfort.
- C.** IF USING THE LORDOTIC INSERTS: Slide the appropriate insert (7°, 10°, 12°) into the elastic pockets on the inside of the slim panel. Adjust for comfort.
- D.** Position the anterior panel over the center of the patient's abdomen with bottom of the panel just above the pubic bone.
NOTE: The patient will need assistance holding the anterior panel in position in order to complete steps E & F!
- E.** Position the compression panels, with the slim panel chairback attached, behind the patient's back. The pull-handle will be on the patient's left side.
- F.** Wrap the left side (with pull-handle) of the brace around the waist and secure it to the anterior panel by attaching the hook strip on the panel to the corresponding loop strip on the inside of the brace.
- G.** Next, wrap the right side around the waist and fasten.
- H.** To apply compression, pull the (pull-handle) away from the body until the desired level of compression is achieved, fasten the pull-handle to the contact strip.

Use and Care:

Hand wash in cold water using mild soap. Rinse thoroughly, dry with towel then air dry completely. If not rinsed thoroughly, soap residue may cause skin irritation and material breakdown.

Warning:

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. If any fraying or deterioration of the cording is noticed, discontinue use.

Notice:

While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability, and comfort, this device is only one element in the overall treatment program administered by a medical professional there is no guarantee that injury will be prevented through the use of this product.

Notice:

This product does not contain latex.

Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months. The lacer component and cord are guaranteed for the lifetime of the brace.

Caution:

Federal Law (U.S.A.) restricts this device to sale by or on the order of a licensed health care professional. For single patient use only.

