

topsheitorthopedics.com  
Phone: 866.592.0488 • Fax: 209.834.8832  
Tracy, CA 95304  
1851 East Paradise Road, Suite A



## **Air Horizon Walker**

### Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.



# Air Horizon Walker

## Instructions for Use

Please read these instructions completely and carefully before applying this product. Correct application is important for proper function of this brace.

**Indications:** Soft tissue injuries of the lower leg, acute ankle sprains, stress fractures of the lower leg, stable fractures of the foot and ankle, stable delayed union or non-union fractures of distal tibia and fibula, achilles tendon repairs.



### Application of Walker (Fig. A)

1. Open the boot liner without removing it from the boot.
2. Place the boot flat on the floor and put your foot inside the boot. Make sure your heel is touching the back of the boot.
3. Secure the boot lining to your leg by overlapping the lining on top of your foot and at the front of your lower leg.
4. Secure the remaining Velcro straps, beginning with the straps closest to your toes (Fig. A).



### Internal Air Pressure Control (Fig. B)

1. To put air inside the boot, turn the valve clockwise until secure, then squeeze the grey bulb until you feel adequate pressure.
2. To remove air from the boot, simply turn the valve counter clockwise.

### Removal of Walker

1. Open the boot liner by detaching all of the Velcro straps.
2. Without removing the liner from the base of the boot, take your foot out of the walker.

### Use and Care:

Hand wash the liner with mild soap and water and lay out to dry. Do NOT machine dry.

### Notes:

For single patient use only.

### Warning:

This product is to be used under the supervision of a medical professional. This device is not intended for unsupervised public use. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

### Warranty:

Top Shelf Orthopedics guarantees this product to be free of defects in material and construction for a period of six months from the date of purchase.

### Caution:

Federal Law (U.S.A.) restricts this device for sale by or on the order of a licensed healthcare professional.